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Instruction for use OlaCare

Read the instructions carefully before use, it contains important data.

Keep these instructions for use in a safe place for future reference.

Description:

This is a diaper-rash cream that is designed primarily to prevent skin irritation in places with a risk of sores by acting as a barrier between the skin and the faeces. It contains Panthenol (in a concentration of 1%), which stimulates the natural regeneration process of the skin after irritation and emulsifier based on the extract of natural proteins of *Solanum Tuberosum*, which improves the barrier function of the cream.

Intended purpose:

The medical device OlaCare is intended for baby's and infants, for prevention of skin irritation in the diaper area. By applying to the diaper areas, it forms an occlusive lipophilic surface layer on the skin that fulfils a barrier function, reduce skin irritation and friction in the affected areas. Panthenol helps to regenerate the skin in cases where it is already irritated, and emulsifier based on the extract of *Solanum Tuberosum*, improves the barrier function of the cream.

Ingredients:

Aqua, Decyl Oleate, Caprylic / Capric Triglyceride, Glycerin, *Solanum Tuberosum* Callus Culture Extract, Cetearyl Alcohol, Methylpropanediol, Petrolatum, Glyceryl Stearate SE, Glyceryl Stearate, Polyacrylate Crosspolymer-11, Panthenol, Dimethicone, Caprylyl Glycol, Sodium Stearoyl Glutamate, Xanthan Gum, Phenylpropanol, Citric Acid.

Indication:

The cream is intended for daily diaper care to prevent diaper rash. Apply only to areas where there is a risk of diaper rash.

Contraindication:

The cream is not intended for the treatment of injured skin.

Interaction:

Undesirable interactions are not yet known for any component of the composition.

Side effects:

The cream is tested to be hypoallergenic. Yet, in exceptional cases, the cream may cause an allergic skin reaction.

Use and method of application:

After thoroughly cleansing the skin in and around the diaper area, apply a thin layer of cream each time you change the diaper. Pay special attention to the folds of the skin. FOR EXTERNAL USE ONLY.

Packaging:

Tubes contains 40 g or 15 g of cream. The sample sachet contains 7g of cream.

Notice:

- Do not use if the package is open or damaged or heavily soiled.
- Carefully close the tube after each use to prevent contaminating the cream.
- Store in a dry place below 30 °C.
- In case of contact with eyes, rinse thoroughly with water. If irritation continues, seek medical attention.
- Keep out of reach of children.

⚠ Warning:

Do not use the cream after the expiry date which is stated on the carton.

Manufacturer: Eurodrug Laboratories B.V., 11 Regulusweg st., The Hague, 2516 AC, The Netherlands, www.eurodrug.nl, Tel.: +31 (0)70 416 73 73

Authorized representative in Ukraine:
Medical HUB LLC
16 Sergey Danchenko st., office 32, Kyiv, 04078
Tel.: +38 (044) 232 52 52, Email: uarep@medical-hub.info

Exclusive distributor in Ukraine:

Megacom LTD
195 B Klochkivska st., Kharkiv, 61145
Tel.: +38 (057) 701 37 52, Email: megacom@megacom.com.ua

Edition 2, revision date 09.07.2021